TAPS & SHOWERING

Construction: Powder Coated with HD Polyethylene Seat

Finishes: White, Clean with warm water and a soft cloth. Do not use harsh abrasives or other household cleaning products.

Installation: Must be permanently fitted to a wall before use. Featuring 8mm diameter holes, it should only be fitted by a competent person and fastened to a wall which has the capacity to withstand the vertical load and leverage applied during use. Fas-

tenings are not supplied because the method of fixing will depend upon the material and thickness of the wall. The fitter should be able to supply suitable fastening.

quired position and reinsert the E clip. Repeat with other leg ensuring both legs are set to the same height

To set Leg height pull out E clip retaining the leg. Slide the leg to re-

Maximum Weight: 160kg



WALL MOUNTED SHOWER SEAT (150kg) WHITE SEA/798/C



The information contained on this page was correct at the date of issue. Fitting dimensions are provided as a guide only. Some variation may occur due to manufacturing tolerances. We pursue a policy of continuing improvement in design and performance of our products and so reserve the right to change specifications without prior notice